

OSTOMY BELT HISTORY

Len Millman developed The Celebration Ostomy Support Belt.

Len had lived with Ulcerative Colitis since he was fourteen years old. In his late forties his situation worsened and he received a Colostomy.

At the time Len was a registered pharmacist living in Las Vegas. He was aware of the products available to him as an ostomate. He was active in golf and tennis and was living as a bachelor in one of the playgrounds of the world. He knew he had to create a belt that would allow him the freedom to enjoy his active life style. He developed the concept for the Celebration Belt fairly rapidly. He knew what he wanted the belt to do for him and he quickly came up with a viable design. He wanted to stop the pendulum action of the bag, he wanted to secure the wafer against his stoma, making it virtually leak proof, he wanted to protect the stoma from trauma and he wanted to be able to conceal the bag and wafer when he felt like it. The Celebration belt was appropriately engineered and produced and was granted a patent for its unique features not available in any other ostomy device.

In 1998 Len passed away.


We know this is an excellent product and are proud to bring it to the ostomy community. It is a tribute to Len that we can move this forward and make it available to all Ostomates. This belt is appropriate for any ostomate who wishes to enjoy an active life style, whether it be sports or simply getting around the house and taking walks and being unafraid to move about.

Celebration Abdominal Catheter Support Belt

The Celebration Abdominal Catheter Support Belt allows you easy access to your Medication Port, Feeding Port or Peritoneal Dialysis tubing.



The CAC Support Belt protects your catheter from catching on your clothes. With a front opening velcro flap or a pocket for your dialysis tubing you will get the easy- to -use support you need.

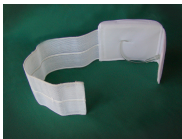

Made of comfortable, washable tricot fabric and elastic this belt will keep you cool and comfortable.

Each CAC belt is custom made to suit your need.

Contact us for your special fit at the address on this brochure.

ORDERING INFORMATION

The Celebration Ostomy belt may be ordered through the following catalogs:

Torbot Ostomy & Medical Supply
1867 Elmwood Ave.
PO Box 3564
Cranston, RI 02910
800-545-4254
fax: 401-780-8740
Order online: www.torbot.com

Look for our advertisement in the United Ostomy Association quarterly magazine and the WOCN quarterly magazine.

Price \$110.00 plus shipping
Medicare approved HCPCS Coding A4396

Retailers interested in offering the Celebration Ostomy Support Belt system please contact:

The Celebration Inc.
PO Box 85
Worthington, MA 01098
413-539-7704

Jack M. Millman, Representative

email: jmmcjf@crocker.com

<http://www.celebrationostomysupportbelt.com>



Copyright © 2007 The Celebration Inc., all rights reserved *The Celebration Ostomy Support Belt is protected under U.S. Patent 5,653,701 Unauthorized use of this name or any part of this ostomy belt will be punished to the fullest extent of the law.*

THE CELEBRATION OSTOMY SUPPORT BELT

Comfortable – Discreet – Virtually
Leak Proof



Medicare Approved

The Celebration Inc.
PO Box 85
Worthington, MA 01098
413-539-7704

Jack M. Millman, Representative

email: jmmcjf@crocker.com

<http://www.celebrationostomysupportbelt.com>

THE NEW SUPPORT BELT

FOR OSTOMATES



The Celebration Ostomy support Belt will allow you to lead an active life.

The Celebration is the only ostomy support belt that maintains the weight of your pouch and prevents swaying while allowing it to fill.

With the Celebration you use your regular pouch and wafer.

The built in support plate provides support for your stoma and reduces the possibility of leakage.

The stoma shield protects your stoma from irritation or trauma.



The Celebration allows you to fold away the empty pouch. No swaying pouch to interfere with athletic activities and intimate moments.



The Celebration Ostomy Belt fits comfortably under clothing allowing you to be confident in your everyday active lifestyle.

FEATURES

The Celebration Ostomy Support Belt is an elegantly simple design. Here's how to wear and care for it.

The Celebration belt is made of a comfortable composite material designed for long life and easy care. Your belt should be hand washed in warm water with a mild detergent and hung to dry.

The plastic Hernia plate and the plastic stoma shield can be cleaned with a mild detergent and wiped dry.

To wear the belt you will first affix your adhesive stoma wafer and pouch over your stoma. In a two part pouch system attach the pouch to the wafer before affixing it over your stoma.

Put the hernia plate into its pocket on the belt.



Place the belt opening in front of the wafer and pull the pouch through the opening. Pull the belt ends around your body to a comfortable snug fit. Secure the belt with the Velcro ends.



The stoma shield may be fitted into the belt over the belt opening to protect the stoma from irritation or trauma.



Fold the parts of the pouch-stabilizing pocket around the pouch. The belt may be worn this way to reduce the pendulum effect of the pouch when contents are in it.



When the pouch is empty you may fold the pocket up and secure it with the Velcro strip at the top of the belt.

SIZING

The Celebration belt has a standard opening of 2 3/8 inches. Opening of 2 3/4 inches is available on request. This will accommodate most standard pouches used by Ostomates. The belt can be comfortably applied on either the right or left side and is appropriate for Colostomy, Ileostomy and Urostomy Ostomates.



To Measure for the proper size belt take a measurement around the body at the level of the stoma while lying on a flat surface

Size	Measurement
Small	27" – 30"
Medium	31" – 34"
Large	35" – 39"
X Large	40" – 45"
XX Large	46" – 51"
XXX Large	52" – 56"